




Instructions and care for your Carbon Fiber AFO

IMPORTANT


Your AFO should only be worn with socks and shoes.

Donning your AFO

- 1 - Remove the shoe's insert and place AFO inside shoe
- 2 - If desired, slide the shoe's insert or custom insert on top of the footplate
- 3 - In a seated position slide foot inside and secure using shoe's velcro or string closures
- 4 - Secure velcro closure on cuff snugly around calf

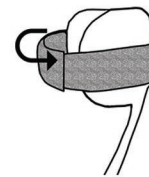

Noodle &
Noodle Classic

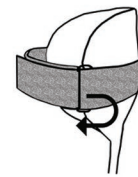

Noodle TA

Doffing your AFO

In a seated position, unfasten shoe's closures first then unfasten velcro on cuff and gently slide foot from shoe.

Caring for your AFO

The cuffs and straps can be handwashed with mild soap and water. Replacement cuff kits are available for re-order through your Orthotist.

For more information on Kinetic Research products, go to
KineticResearch.com

or write to us at
email@kineticresearch.com

5513 West Sligh Avenue Tampa, Florida 33634
1-800-919-3668 KineticResearch.com